



information. innovation. infection control.

stable-LINE® | Arterial Catheter Stabilization Device



Instructional Technique

U.S. Patent No. 6,827,707

12675 Stowe Drive, Suite B
San Diego, CA 92064

Tel: 888-575-6341
Fax: 858-391-2759
www.iapyx.com

information. innovation. infection control.

Iapyx Medical uses innovation and experience to develop products that protect clinicians and patients from hospital-acquired infections.

stable-LINE® | Arterial Catheter Stabilization Device


Application Technique




Cleanse and degrease insertion and target stabilization site, insert catheter, and apply transparent dressing according to hospital policy. Remove center paper backing from the Stable-Line Arterial device.




Align the Stable-Line device over the extension set tubing, behind the Luer as shown. Gently press tubing into the stabilization notch.

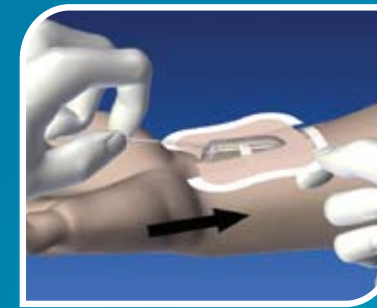

Slide the Stable-Line device forward until the shield stops behind the male Luer connector. The Luer, catheter and insertion site should be confined under the protective shield.

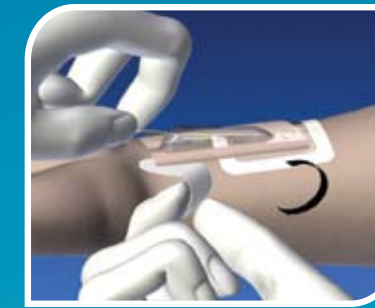


Remove each side tab separately and gently press down on the 3M™ adhesive pad to stabilize arterial catheter.

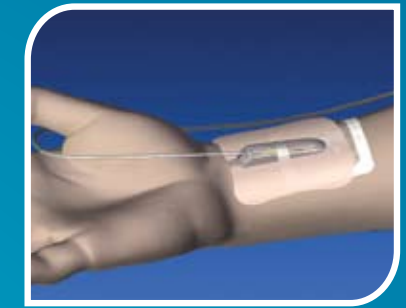

Catheter protected and stabilized.

Removal Technique

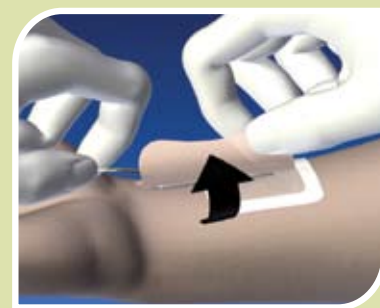

With one hand, stabilize the extension set tubing. With other hand, gently lift one side of the 3M adhesive pad and disengage tubing from stabilization notch.



Continue to gently remove adhesive pad from site. Remove transparent dressing and complete site care according to hospital policy if performing a dressing change.



Reapply another Stable-Line Arterial device according to the Application Technique if not discontinuing arterial catheter.

Change when clinically indicated, at least every 7 days.

DISTRIBUTOR INFORMATION :